


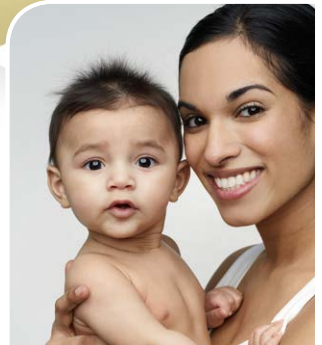

# Swing™

*No. 1 choice  
of hospitals  
& mothers*

## EN Troubleshooting Guide |



## 10. Troubleshooting

Problem	Solution
<b>If the motor is not running</b>	<p>Check presence of mains current.</p> <p>Check the position of the batteries in the battery compartment.</p>
<b>If there is low or no suction</b>	<p>Verify the valve head and membrane are clean and not damaged.</p> <p>Ensure the membrane is attached correctly to the valve head.</p> <p>Check the lateral position of the valve.</p> <p>Check all connections.</p> <p>Check the ends of the tubing fit securely into the back of the connector and into the port on the breastpump.</p> <p>Do not kink the tubing during expression.</p> <p>Ensure the breastshield forms a complete seal around the breast.</p>
<b>If condensation appears in the tubing</b>	<p>Rinse out the tubing.</p> <p>Shake out any water droplets.</p> <p>Hang up the tubing to air-dry.</p>
<b>If milk gets into the tubing</b>	<p>Turn off the pump and unplug from power source.</p> <p>Remove and clean the tubing.</p> <p>Shake out water droplets and hang to air-dry.</p>
<b>If milk gets into the pump</b>	<p>Turn off the pump and unplug from power source.</p> <ol style="list-style-type: none"> <li>1. Open the battery cover, remove the sponge. Wash the expanded-plastic liner with clear, lukewarm water and squeeze out well. Allow to dry thoroughly.</li> <li>2. Pour about 200 ml of warm soapy water into a container.</li> <li>3. Connect the tubing and the power cable onto the pump.</li> <li>4. Hold the device with the control button upwards over a basin and switch on. Press  and ensure that the lowest vacuum level is set by pressing  twice.</li> <li>5. Hold the end of the tubing that is not connected to the pump in the container with the soapy water.</li> <li>6. The water is pumped through the tubing into the device. Allow it to pump until all the water has been flushed through.</li> <li>7. Repeat procedure with 100 ml of clear, warm water.</li> <li>8. Then leave the device switched on for 3 minutes, until the pump is completely dry.</li> <li>9. As soon as the expanded-plastic liner is dry, put it in place again and close the battery cover.</li> <li>10. If necessary, clean the pump set according to cleaning instructions „Prior to first use and once daily“.</li> </ol>